

B-02

Microdermabrasion Instrument **INSTRUCTION**




Dear customer,

Thanks for your choosing our products.

For the best capability and safety, please read the manual before operation.

Please reserve the manual for your reference.

Panel and Accessories

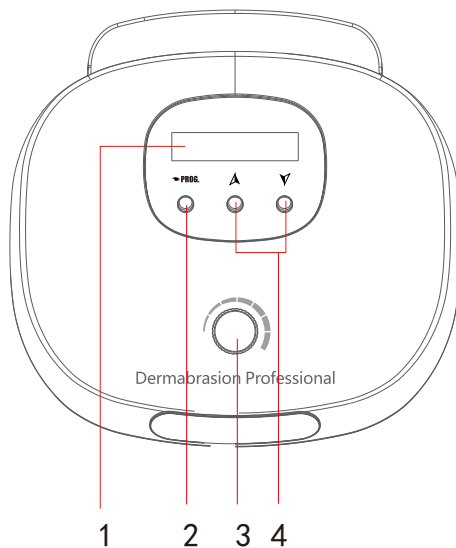


1.Menu Display

2.Menu


3.Suction Adjust knob

4.Time Adjust

■Diamond Microdermabrasion Accessories

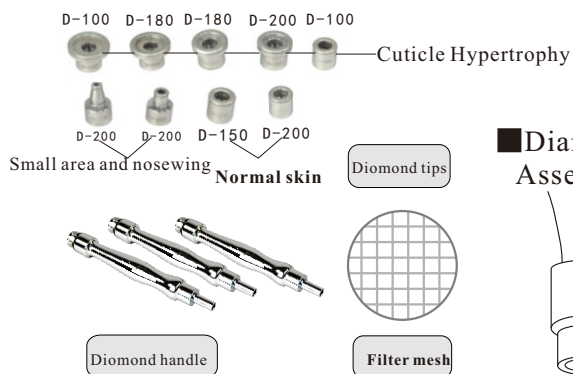

■Diamond Microdermabrasion Assembly

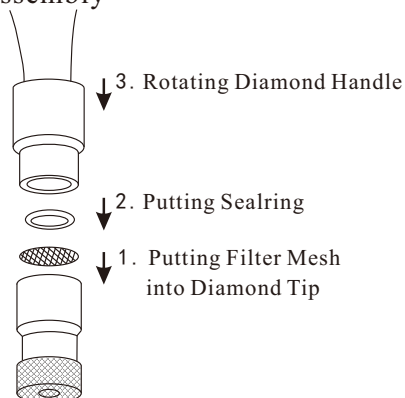

Diamond Dermabrasion Function Instruction

Diamond Dermabrasion Principle

Microdermabrasion is a skin-freshening technique that helps repair facial skin that takes a beating from the sun and the effects of aging. The plastic surgeon uses a device like a fine sandblaster to spray air together with the diamond tips across the face, mixing gentle abrasion with suction to remove the dead, outer layer of skin. As with other skin rejuvenation techniques, more than one treatment may be needed to reduce or remove fine wrinkles and unwanted pigmentation.

Diamond Dermabrasion Function

Microdermabrasion is effective in reducing fine lines, "crow's feet," age spots and acne scars. It stimulates the production of skin cells and collagen. It has proven to be a very popular nonsurgical cosmetic procedure.

When used regularly, this system works wonderfully on all skin types can be used on the face, neck, decollate or the entire body for a youthful, polished appearance. You will notice immediate visible results.

After the treatment, you should use brush with water to clean diamond heads, after the cleaning, put it into the solution consist of 75% alcohol for 15 minutes, then take it out and wipe it to complete dry. Put it into the ultraviolet radiation sterilizer for next using.

Diamond Dermabrasion Function Instruction

ADVANTAGES

1. Flexible adapts to every microdermabrasion machine no matter what brand you have. Easy for installation, function and solve the clogging problem for powers.
2. Precision Nine different types of tip shape and diamond crystals sizes, which used in different part of body, even the eye section. Selecting the appropriate tip number, along with adjustable vacuum, allows for controlled abrasion.



Full body microdermabrasion before slimming program, acne armed, cellulite etc.



Eye section especially for wrinkles, blackhead, whiteheads etc.



Scars, fine lines, sun damaged skin focused on the bigger treatment

3. Durability The diamond surface allows for an extremely long life.
4. Safety

By eliminating aluminum oxide crystals, there is no risk of particles remaining embedded in skin. Neither patient nor clinician has to breathe aluminum oxide dust, and the treatment room remains clean. This non-invasive skin care treatment can be completed in a short -period of time and without pain. This skin care treatment is ideal for people on the go. There is no downtime from work or social activities.



5. Result after operation:

When the outer layer of dead and damaged skin is removed and the skin is clean and smooth, the action of skin care product is greatly accelerated.

Diamond Dermabrasion Function Instruction

6. Curing the following symptom:

- Acne
- Pigmentation
- Scarring
- Stretch Marks
- Wrinkle

Procedure

1. Steaming
2. Microdermabrasion treatment
3. Washing face cream clean skin
4. Face mask for whiten and water
5. Esssence

Caution of Treatment

Caution of interrelated products

1. The cosmetic that you use together with the machine should be with the high quantity and safe function.
2. Don't use the cosmetic with the element of alcohol, acid and alkali.
3. It's necessary to use the moisture and collagen products to protect the skin.
4. After the treatment, please use the sun block cream every day.
5. It will be helpful if use the essence gel, but please use the gel with anti-irritability and diminishing inflammation function

Taboo:

1. This instrument is very professional and only the beautician or doctor can operate it.
2. The people with skin problems can not use it.
3. The people with capillaries expansion can not use it.
4. This appliance is not intended for use by person (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
5. Children should be supervised to ensure that they do not play with the appliance.

Diamond Dermabrasion Treatment Process

1. Connect the wire with the mainframe.



2. Press adjust the vacuum by turning this knob.



3. Select the appropriate diamond tip to connect with mainframe and put a specialized filter mesh into the diamond tip

Note :

(A) If feel suction too small, you have to re-replace filter during the operation. The number of different filters must be obtained by personal skin thickness of different areas and different sizes, each operation must be re-filled filters, after the operation the peeling off Skin can be taken to the customer

((B) Please check whether the filters are in the right place, you can paint some solid lubricant into the metal joint of the straw, such as Vaseline, etc.


(C) Microdermabrasion tips are of different size. Using tactile and visual sense to make sure what's a regional hypertrophy, generally, first choose the rough one to operate, and then the fine one.

4.. Suction adjusts. The thin cortex, the lower the suction, you should set the lowest suction for the eye bag, crow's-feet etc.

5. Press button to adjust time as 5/6 minutes.

Diamond Dermabrasion Treatment Process

6.. You should test the suction by arm or hand before operation. Get an average suction on the surface area. Be careful and don't leave the diamond tips over any area for more than 15seconds. Otherwise, high pressure will lead to swelling or bleeding. Natural skin with direct line, and for the sensitive skin, operates as the picture A shows

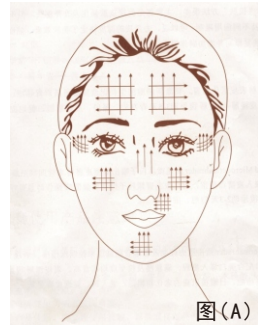

7. After the operation, should use brush with water to clean diamond heads, after the cleaning, put it into the solution consist of 75% alcohol for 15 minutes, then take it out and wipe it to complete dry. Put it into the ultraviolet radiation sterilizer for next using.

Technical parament

Specification

Power Supply: 220V

Frequency : 50Hz

Power : 10W

Dimensions :

Table Type : 37. 5×31×26CM

Electric Current:2A

N. W: 2.18kg G.W.:2. 98 KG

Packing list

Item Name	Unit	Qty	Remarks	
Main frame	Set	1		
Microdermabrasion accessories	Set	1		
Microdermabrasion tube	Set	1		
Filter acrylic spanner	Pc	1		
Power wire	Pc	1		
Fuse	Pc	2		
Accessories box	Pc	1		
Instruction	Pc	1		

Important Safety Notice

1. Please don't disassemble the equipment or try to do some other operation, which have not been indicated in our instruction. All the repairing work should be done by our admitted professional personnel
2. Please don't assemble or operate this equipment while you are near the water or your hands are wet. Don't pour the liquid on it.
3. Please pull out the plug from the socket when you met such conditions as below. And contact professional maintenance Station.
 - a. The equipment touched liquid
 - b. The equipment has abnormal smell, smoking or loud and strong noise.
 - c. Cable is broken
 - d. The equipment is fallen down and broken
4. Don't put anything on the cables; don't place the equipment at any place where someone can step on the cable.
5. Please pull out the power cable after using it for your safety
6. Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately
7. Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.
8. Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.